


## Carpus/Wrist Pain or Injury – Acute

Acute hyperextension injury, acute inflammation with chronic hyperextension or arthritis, autoimmune mediated polyarthropathy with carpal effusion, carpal strain, flexor carpi ulnaris strain, effusion at the carpus secondary to tick borne disease, post carpal arthrodesis.



1. A small Assisi Loop can be used to treat the carpal joint/wrist in small to medium dogs. A large- or giant-breed dog should use a large Assisi Loop. If both carpal joints/wrists are to be treated at the same time, then a large Loop should be utilized. Positioning is extremely important for optimal effect. **See Pictures**
2. Position the pet lying down either flat on the ground or up on their elbows.
3. Identify the carpal joint(s)/wrist(s) and place one or both joints in the treatment area, laying the Assisi Loop over the area. **See Pictures**
4. The signal generator can be positioned to help balance and stabilize the unit during treatment.
5. The signal generator should be located in view of the person treating the pet.
6. Do not leave the pet unattended when using this placement as small movements can change the positioning of the Loop and the area being treated.
7. Turn the unit on by pressing the button on the front of the white rubberized signal generator; green lights will illuminate about once per second.
8. Keep the unit in place for the full 15 minutes until the unit shuts off.
9. Use the unit 4 times daily, if possible, for 3-5 days. Treat twice daily for an additional 5 days or until pain is completely resolved. Then use as needed.
10. There should be a minimum of 2 hours between treatments for optimal nitric oxide enhancement.

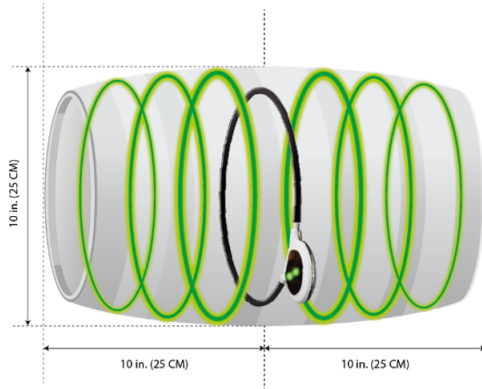

**Effective Treatment Zone (ETZ):** The 20 CM Assisi Loop has been designed to deliver a clinically effective therapeutic signal to damaged tissue that is positioned within the perimeter of a barrel-shaped treatment zone as illustrated.

**Pet Owner Responsibility.** A panel of licensed, experienced veterinarians has developed this Guide. Pet owners should use this protocol only to treat a condition listed above and as directed by a qualified veterinary clinician. Other protocols in this Guide should be used only as directed by a qualified veterinary clinician.

**Important:** Before starting treatment, please consult product label for optimal product use and safety guidelines. To learn more about the technology, visit our website.

**Battery Life:** When the lights are blinking 2-3 times per second, the battery is running low and a new Assisi Loop should be purchased. There should be a minimum of 2 hours between treatments for optimal battery efficiency.