


Dilated Cardiomyopathy



1. A large Assisi Loop should be used to treat the chest of the pet.
2. Position the pet lying down either flat on the ground, up on their elbows, sitting, or being held.
3. If the pet is small enough, the Assisi Loop can be placed around the chest of the pet.
4. Placing the Assisi Loop over the head:
 - a. Place the Assisi Loop over the head of the pet with the signal generator toward the ground. Place the top of the Loop at the deepest part of the chest; this will allow proper treatment as well as help hold the Loop in place. **See Pictures**
 - b. If the pet is head shy, cover their eyes while placing the Loop over their head.
5. Medium- to large-sized pet – The Assisi Loop can be placed on the side of the chest over the area that is to be treated. **See Pictures**
6. Giant- or wide-breed dogs (Rottweilers, Mastiffs, Newfoundlands, . . .) – Two Assisi Loops can be used simultaneously to treat both sides. One Assisi Loop can be used on the two sides consecutively, but there should be 2 hours in between sessions for optimal battery longevity.
7. An Assisi LOOP-AID can be used and the Assisi Loop attached to the garment with two Velcro straps to maintain the Loop position with the center of the Assisi Loop over the chest/lungs/thorax. **See Pictures**
8. The signal generator should be located in view of the person treating the pet.
9. Do not leave the pet unattended when using this placement as small movements can change the positioning of the Loop and the area being treated.
10. Turn the unit on by pressing the button on the front of the white rubberized signal generator; green lights will illuminate about once per second.
11. Treat 4 times daily, if possible, until clinical signs have resolved. Then treat twice daily.
12. There should be a minimum of 2 hours between treatments for optimal nitric oxide enhancement.

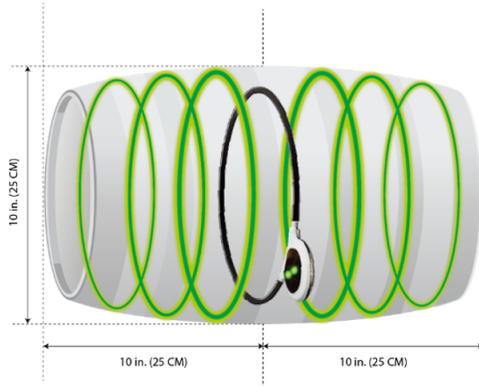

Effective Treatment Zone (ETZ): The 20 CM Assisi Loop has been designed to deliver a clinically effective therapeutic signal to damaged tissue that is positioned within the perimeter of a barrel-shaped treatment zone as illustrated.

Pet Owner Responsibility. A panel of licensed, experienced veterinarians has developed this Guide. Pet owners should use this protocol only to treat a condition listed above and as directed by a qualified veterinary clinician. Other protocols in this Guide should be used only as directed by a qualified veterinary clinician.

Important: Before starting treatment, please consult product label for optimal product use and safety guidelines. To learn more about the technology, visit our website.

Battery Life: When the lights are blinking 2-3 times per second, the battery is running low and a new Assisi Loop should be purchased. There should be a minimum of 2 hours between treatments for optimal battery efficiency.