


Eye Infection/Inflammation

Blepharospasm, entropion, bacterial or viral eye infections (to treat associated inflammation and pain, not infection), conjunctivitis, uveitis, indolent ulcers, eye ulcers, corneal ulcers.



1. A small or large Assisi Loop can be used to treat the eyes in the pet depending on the size of the pet. Dachshund- and Chihuahua-sized pets do best with small Assisi Loops. Larger breeds need the larger Loop. Positioning is extremely important for optimal effect.
2. Position the pet lying down either flat on the ground, up on their elbows, or sitting.
3. Place the Assisi Loop over the head of the pet with the signal generator toward the ground. Place the top of the Loop just behind the pet's eyes or attached to an Elizabethan collar; this will allow proper treatment as well as help hold the Loop in place. **See Pictures**
 - a. If the pet is head shy, cover their eyes while placing the Loop over their head.
4. When attaching the Assisi Loop to the Elizabethan collar, position the Loop so the eyes are in the center of the Loop. Tape the Loop in several places to keep it secure and in place for the whole treatment.
5. The signal generator should be located in view of the person treating the pet.
6. Do not leave the pet unattended when using this placement as small movements can change the positioning of the Loop and the area being treated.
7. Turn the unit on by pressing the button on the front of the white rubberized signal generator; green lights will illuminate about once per second.
8. Keep the unit in place for the full 15 minutes until the unit shuts off.
9. Use the unit 4 times daily, if possible, for 3-5 days or until pain and inflammation is gone.
10. There should be a minimum of 2 hours between treatments for optimal nitric oxide enhancement.

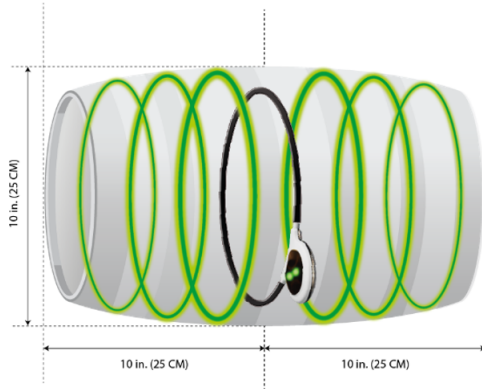

Effective Treatment Zone (ETZ): The 20 CM Assisi Loop has been designed to deliver a clinically effective therapeutic signal to damaged tissue that is positioned within the perimeter of a barrel-shaped treatment zone as illustrated.

Pet Owner Responsibility. A panel of licensed, experienced veterinarians has developed this Guide. Pet owners should use this protocol only to treat a condition listed above and as directed by a qualified veterinary clinician. Other protocols in this Guide should be used only as directed by a qualified veterinary clinician.

Important: Before starting treatment, please consult product label for optimal product use and safety guidelines. To learn more about the technology, visit our website.

Battery Life: When the lights are blinking 2-3 times per second, the battery is running low and a new Assisi Loop should be purchased. There should be a minimum of 2 hours between treatments for optimal battery efficiency.