

Gastrointestinal and Organ Issues – Caudal/Lower Abdomen

Inflammatory bowel disease, colitis, chronic diarrhea, cystitis, urinary tract obstruction, fecal or urinary incontinence, hemorrhagic enteritis, uterine inflammation.




1. A large Assisi Loop should be used to treat the abdomen of the pet.
2. The area to be treated will either be the cranial or front half of the abdomen or the caudal or back half of the abdomen. If the area to be treated is the stomach, pancreas, liver, spleen, or gall bladder, the cranial or front half of the abdomen is to be treated. If the area to be treated is the small intestine, large intestine, colon, or uterus, the caudal or back half of the abdomen is to be treated. In a cat or small dog, the whole abdomen can be treated with a large Assisi Loop.
3. Position the pet lying down either flat on the ground, up on their elbows, sitting, or standing. Place the Assisi Loop on the pet's back, side, or around the abdomen. Center the Loop just in front of the pelvis. **See Pictures**
4. If the pet is small enough, the Assisi Loop can be placed around the abdomen of the pet.
 - a. If the pet is small enough, place the pet's rear limbs through the Assisi Loop and slide the Loop over the rear limbs and tail onto the trunk. Place the Assisi Loop just in front of the pelvis. **See Pictures**
 - b. **CAUTION: Do not force the pet's shoulders, haunches, or torso through the coil. If they do not slip easily through the coil, place the coil flat against, over, or under animal's torso, as shown in the related images. Forcing the animal through the coil can be traumatic to the pet and may overly stress the connection of the coil to the circuit board inside the Loop case.**
5. Medium- to large-sized pets – If the pet is too large for the above placement, the Assisi Loop can be placed:
 - a. On the side of the abdomen over the area that is to be treated. **See Pictures**
 - b. With the pet on their back with the Assisi Loop on their abdomen. **See Pictures**
6. Giant- or wide-breed dogs (Rottweilers, Mastiffs, Newfoundlands, . . .) – Two Assisi Loops can be used simultaneously to treat both sides if indicated. One Assisi Loop can be used on

For additional questions, please call or email customer support:

the two sides consecutively, but there should be 2 hours in between sessions for optimal battery longevity.

7. If the pet will not lie down, or will lie down but not stay still, an Assisi Loop-AID can be used. Attach the Assisi Loop with two Velcro straps to maintain the Loop position with the center of the Assisi Loop over the lower abdomen. **See Pictures**
8. The signal generator should be located in view of the person treating the pet.
9. Do not leave the pet unattended when using this placement as small movements can change the positioning of the Loop and the area being treated.
10. Turn the unit on by pressing the button on the front of the white rubberized signal generator; green lights will illuminate about once per second.
11. Keep the unit in place for the full 15 minutes until the unit shuts off.
12. In cases of severe disease (hemorrhagic gastroenteritis, acute kidney failure, urinary obstruction, acute episode of IBD, . . .), use the unit 4 times daily, if possible, until clinical signs have resolved. Then treat twice daily for an additional 3-5 days.
13. In mild or early cases, the Loop can be used 2 times daily for 48 hours or until clinical signs have resolved.
14. There should be a minimum of 2 hours between treatments for optimal nitric oxide enhancement.

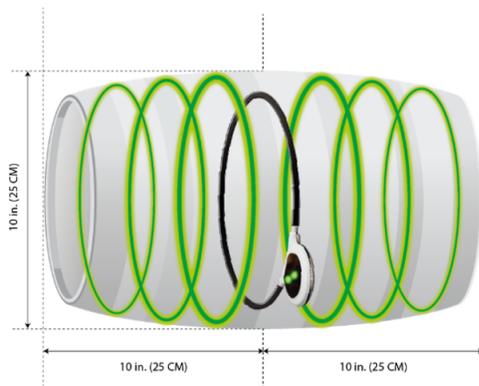

Effective Treatment Zone (ETZ): The 20 CM Assisi Loop has been designed to deliver a clinically effective therapeutic signal to damaged tissue that is positioned within the perimeter of a barrel-shaped treatment zone as illustrated.

Pet Owner Responsibility. A panel of licensed, experienced veterinarians has developed this Guide. Pet owners should use this protocol only to treat a condition listed above and as directed by a qualified veterinary clinician. Other protocols in this Guide should be used only as directed by a qualified veterinary clinician.

Important: Before starting treatment, please consult product label for optimal product use and safety guidelines. To learn more about the technology, visit our website.

Battery Life: When the lights are blinking 2-3 times per second, the battery is running low and a new Assisi Loop should be purchased. There should be a minimum of 2 hours between treatments for optimal battery efficiency.

For additional questions, please call or email customer support:

info@assisianimalhealth.com 866-830-7342
www.assisianimalhealth.com © 2018 AAH Holdings LLC