


## Sacroiliac (SI) Pain or Injury – Acute

Pain on pressure over the sacrum when standing or when the hip is extended and the sacrum is stabilized.



1. A small or large Assisi Loop can be used to treat the sacroiliac area of the pet depending on the size of the pet. Small Assisi Loops can be used for up to Beagle-sized pets when treating just one SI joint. Larger Assisi Loops can be used to treat both SI joints simultaneously on small pets when positioned correctly. Dogs larger than a Beagle should use a large Assisi Loop. With a large Assisi Loop, both SI joints can be treated simultaneously. Positioning is extremely important for optimal effect. **See Pictures**
2. Position the pet lying down either flat on the ground or up on their elbows.
3. When treating **one SI joint** at a time, place the Assisi Loop so that the wing of the Ilium is in the center of the Loop. **See Pictures**
4. To treat **both SI joints** simultaneously, place the Assisi Loop around the pelvis so that the Loop is positioned over both wings of the ilium, the highest points of the pelvis. **See Pictures**
5. If the pet will not lie down, an Assisi LOOP-AID can be used. Attach the Assisi Loop with two Velcro straps to maintain the Loop position with the center of the Assisi Loop over the sacroiliac joint(s). **See Pictures**
6. The signal generator can be positioned to help balance and stabilize the unit during treatment.
7. The signal generator should be located in view of the person treating the pet.
8. Do not leave the pet unattended when using this placement as small movements can change the positioning of the Loop and the area being treated.
9. Turn the unit on by pressing the button on the front of the white rubberized signal generator; green lights will illuminate about once per second.
10. Keep the unit in place for the full 15 minutes until the unit shuts off.
11. Use the unit 4 times daily, if possible, for 5 days. Treat twice daily for an additional 5 days or until pain is completely resolved.
12. There should be a minimum of 2 hours between treatments for optimal nitric oxide enhancement.

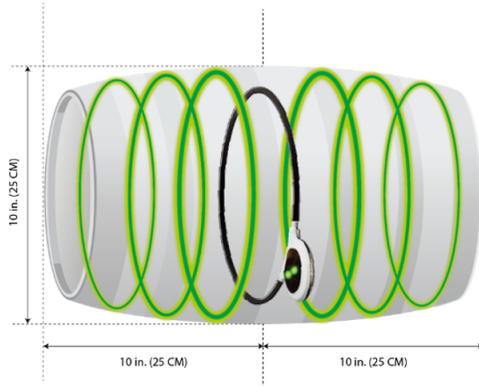

**Effective Treatment Zone (ETZ):** The 20 CM Assisi Loop has been designed to deliver a clinically effective therapeutic signal to damaged tissue that is positioned within the perimeter of a barrel-shaped treatment zone as illustrated.

**Pet Owner Responsibility.** A panel of licensed, experienced veterinarians has developed this Guide. Pet owners should use this protocol only to treat a condition listed above and as directed by a qualified veterinary clinician. Other protocols in this Guide should be used only as directed by a qualified veterinary clinician.

**Important:** Before starting treatment, please consult product label for optimal product use and safety guidelines. To learn more about the technology, visit our website.

**Battery Life:** When the lights are blinking 2-3 times per second, the battery is running low and a new Assisi Loop should be purchased. There should be a minimum of 2 hours between treatments for optimal battery efficiency.