


Severe Pain Related to Arthritis

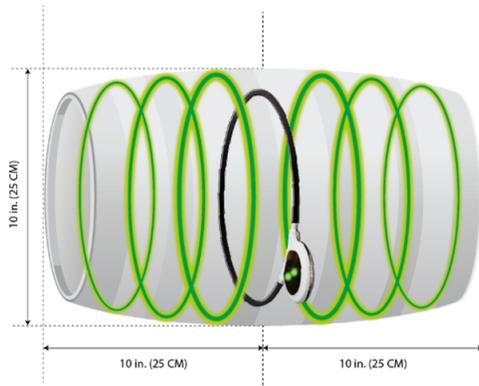

Effective Treatment Zone (ETZ): The 20 CM Assisi Loop has been designed to deliver a clinically effective therapeutic signal to damaged tissue that is positioned within the perimeter of a barrel-shaped treatment zone as illustrated.

1. Locate the joint that is affected in the “Contents” and follow the guidelines.
2. Use the Assisi Loop 4 times daily, if possible, for 10 days or until there is improved mobility and pain is greatly decreased or eliminated. Use of the Loop can be tapered down to 1-2 times daily as the condition improves. Long-term chronic conditions may require as much as 3-6 weeks before tapering would be appropriate. If symptoms resolve and regular use of the Assisi Loop ceases, use the Assisi Loop as needed if pain starts to recur.
3. There should be a minimum of 2 hours between treatments for optimal nitric oxide enhancement.
4. **For those treating osteoarthritis in multiple parts of the body:** The Loop is only guaranteed to last the *minimum* 150 treatments if it is not used more frequently than once every 2 hours: If you are treating OA in multiple parts of the body, you will need to treat EACH AREA at least twice per day. We recommend:
 - Using more than one Loop so you can treat multiple areas during the same 15 minute period
 - Use the same Loop for multiple areas, waiting 2 hours between treatments
 - Use the Loop more frequently than every 2 hours, knowing that the battery may last for *significantly fewer treatments and negate the warranty*

Pet Owner Responsibility. A panel of licensed, experienced veterinarians has developed this Guide. Pet owners should use this protocol only to treat a condition listed above and as directed by a qualified veterinary clinician. Other protocols in this Guide should be used only as directed by a qualified veterinary clinician.

Important: Before starting treatment, please consult product label for optimal product use and safety guidelines. To learn more about the technology, visit our website.

Battery Life: When the lights are blinking 2-3 times per second, the battery is running low and a new Assisi Loop should be purchased. There should be a minimum of 2 hours between treatments for optimal battery efficiency.