
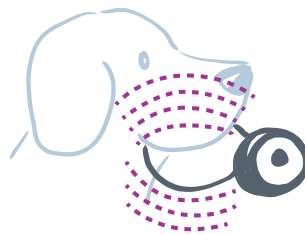


Operating Instructions – Assisi DentaLOOP™

Thank you for choosing the Assisi DentaLOOP™, a non-invasive, non-drug therapy designed specifically for use before and after dental procedures and oral surgery. The Assisi DentaLOOP delivers the same effective signal as the Assisi LOOP® and offers 60, 15-minute treatments. Use only on your pet as instructed by your veterinarian.

1. Place the loop over your pet's muzzle, or adjacent to the area that needs treatment, as directed by your veterinarian.

Note that the DentaLOOP's coil should be positioned within 4-5 inches of the area to be treated (i.e., so that the area treated is within the treatment zone). Thus, if a large dog is being treated after molar extractions, the coil should be positioned toward the back of the mouth/jaw, with the battery case toward the front of the jaw (nose).

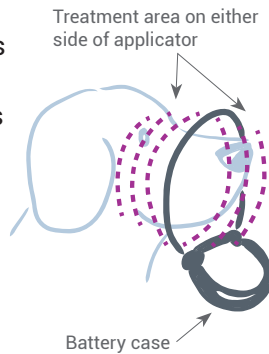

2. Press the on/off button once. A small light will flash on the side of the case indicating treatment is being delivered. Treat for 15 minutes.
3. The device will automatically turn itself off after 15 minutes.
4. If you want to treat for less than 15 minutes, simply press the on/off button to turn the device off.
5. For optimal battery life, wait at least 2 hours between treatments unless your veterinarian prescribes more frequent use, (note: battery life could be reduced substantially, thus rendering fewer than 60 treatments).

See protocol recommendations on page 2 of this document.

Safety Warnings

- Equipment NOT SUITABLE FOR USE IN THE PRESENCE OF FLAMMABLE ANESTHETIC MIXTURE WITH AIR or WITH OXYGEN or NITROUS OXIDE or OXYGEN ENRICHED ATMOSPHERES.
- Pulsed electromagnetic field therapies may have effects on the functioning of powered, metallic cardiovascular implants or neuronal implants (e.g., pacemakers). Make sure that distance is maintained from individuals with these types of implants while Assisi DentaLOOP is delivering treatment.
- Not protected for immersion in water. Do not get the Assisi DentaLOOP wet.
- Do not use unless case and applicator loop are in good condition.

Applicator Etiquette

It is important that when the DentaLOOP is in use, the battery case be positioned away from the coil at either a right angle or flat and in the same plane. Folding the battery case within the Loop will negate the signal, and the device will not deliver the optimal effective therapy.



Add the convenience of hands-free healing with the DentaLOOP™ Bonnet (sold separately).



There are no user serviceable parts inside the Assisi DentaLOOP.

How Often to Use the Assisi DentaLOOP™:

It is recommended to start with two to four 15-minute treatments per day, spaced at least two hours apart, for pre-surgery, and post-surgery or post-trauma conditions.

- Pre-Surgery conditions: Use DentaLOOP around the pet's injured, inflamed, or painful area of the mouth. Two 15-minute sessions per day help reduce pain and inflammation, and improve affected tissue in preparation for a dental or orofacial procedure.
- Post-Surgery or Post-Trauma Protocol: Apply the DentaLOOP around the patient's muzzle or adjacent to the injured or affected area two to four times per day for 15 minutes to help reduce edema and pain, and hasten healing.

Ultimately, you want to consider that the Assisi DentaLOOP is catalyzing the anti-inflammatory cascade as well as accelerating the body's own healing properties. Please rely on your veterinarian to assess the animal and to prescribe the best treatment protocols for each case.

How soon will I see results?

Depending on the severity and duration of the condition being treated, you may see results as early as after the first treatment or two, or it may take up to a week or more of treatments to notice results.



To view videos on how to use the
DentaLOOP and the DentaLOOP Bonnet, visit
assisianimalhealth.com/DentaLOOP-inst